



WandScan 900 Pro Scanner

AKC Reunite WandScan 900-Pro has powerful microchip detection capabilities that help keep the user at a safe distance from uncontrollable, dangerous or unknown animals. Round scanner antennae have two read areas to help check for a chip twice in one pass, making scanning easier and faster.

Scanning Instructions

- 1. Insert battery prior to use.
 - Press the red button to turn the scanner on it will beep and the LCD screen will light up when the sensors are activated. Extend pole with the loop of the "scanning" wand pointed toward the pet. The scanner antennae are located at the "3 o'clock" and "9 o'clock" sides of the scanner's loop.
 - Slowly scan the area between the shoulder blades in a zigzag motion.
 - If no microchip is detected between the shoulder blades, scan a broader area, including the front legs down to the elbows and the neck and chest.
- 2. When a microchip is detected, the scanner will beep and the screen will display the pet's microchip number.
- 3. To help reunite a found pet with its owner, go to www.petmicrochiplookup.org or call AKC Reunite at 800-252-7894.

The scanner will beep twice and automatically turn itself off in approximately 30 seconds.

For future reference: if the scanner is working slowly, replace the 9-volt battery.

For best results, remove metal collars before scanning. Computers, some automobile key FOBS, metal tables and even fluorescent lighting can interfere with effective scanning.

WARNING: Products with exposed solder may contain lead, a chemical known to the State of California to cause birth defects and other reproductive harm. Please wash hands after handling internal components and motherboards and avoid inhalation of fumes if heating the solder.

DO NOT HANG THIS SCANNER ON A HOOK!

Please lay this item flat when storing. If this scanner is hung by its cord, the antenna will be displaced and significantly impact the effective scanning of pets with microchips.